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
型号：ST-T1-BLT-V

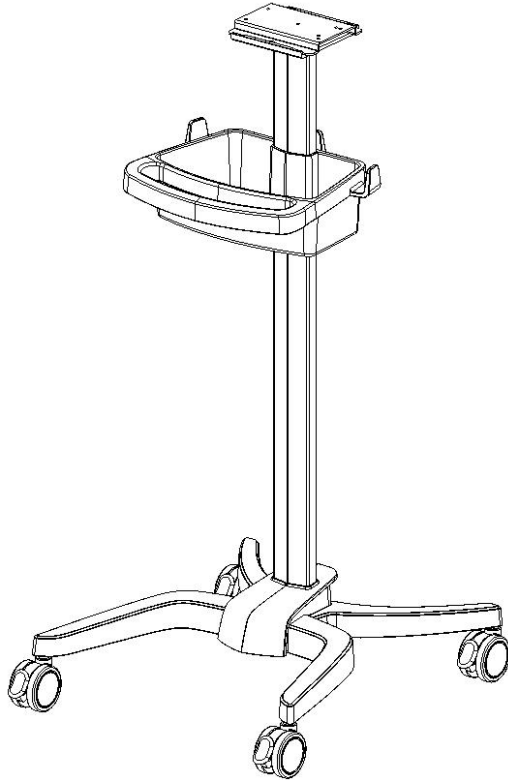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
Roll Stand Installation Guide

Model: ST-T1-BLT-V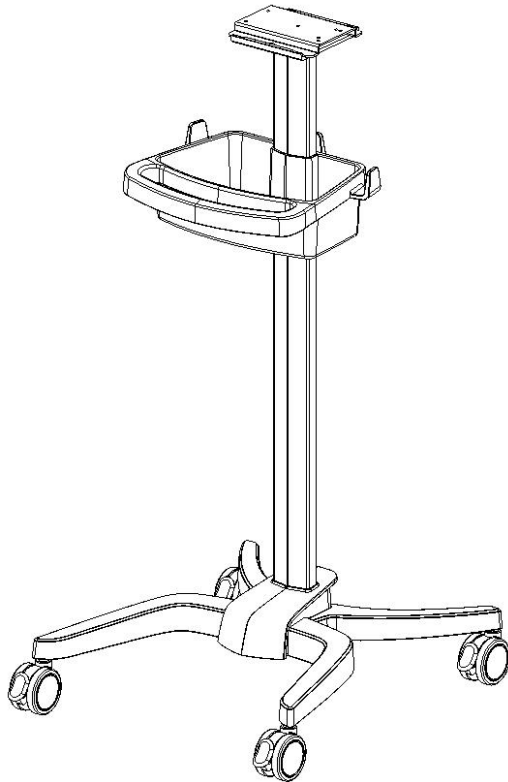

Note: This installation guide suitable for Biolight V series patient monitor

1 Specification


Hardware specifications

Model	ST-T1-BLT-V
Photo	
Load Capacity	7.5kg as Standard
Angle of pitch	0° Fixed as Standard
Rotation angle	360°
Trolley Size	446mm*483mm*956mm
Height	956mm
Basket	Plastic With handle (267mm*245mm*100mm)
Caster	2.5inch, All four wheels have brakes
Lifting method	Can't lift
Material	Cart: aluminum alloy - powder spraying, plastic - primary color injection molding. Mounting plate: aluminum alloy - anodized
Package Size	770mm*480mm*195mm
Net weight	6.8kg
Gross weight	8.5kg
Certificates	ISO9001




2 Packing List

NO.	Description	Qty.	Note
1	Base (including casters)	1	With 4 brakes
2	Pole	1	
3	Mounting plate	1	The mounting plate of different monitors may be different
4	Accessory basket	1	
5	Accessory bag	1	Tools and screws

3 Roll Stand Parts Schematic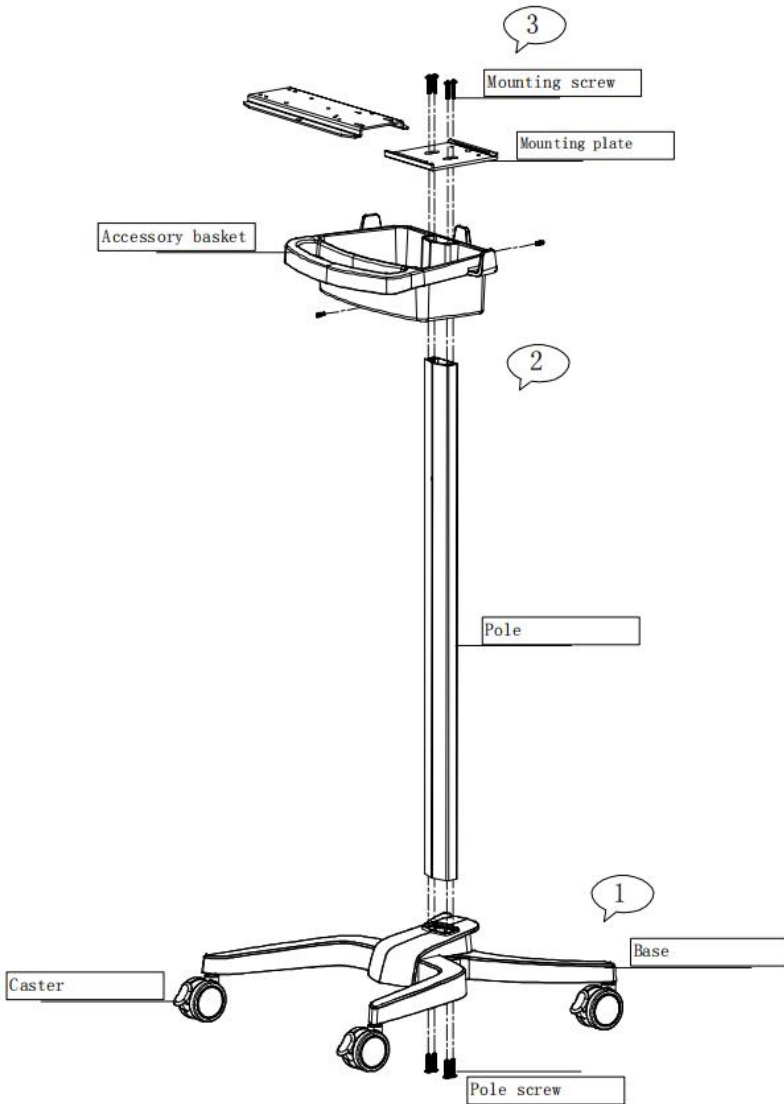

4 Assemble the roll stand
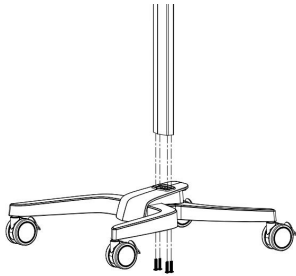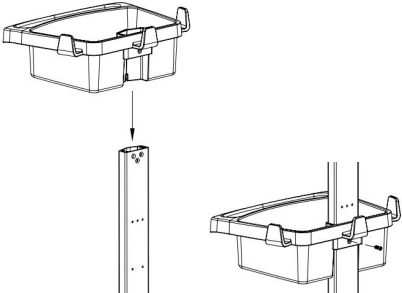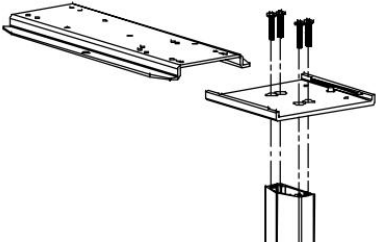
<p>1</p>	<ul style="list-style-type: none">- Insert the column into the base- Fix the post to the base with four hexagon screws	
<p>2</p>	<ul style="list-style-type: none">- Clamp the attachment basket through the post to the post screw- Fix the accessory basket to the post with one hex socket screw	
<p>3</p>	<ul style="list-style-type: none">- Secure the mounting plate to the post with 4 countersunk screws	

5 Cleaning and Sanitizing

Carts can be cleaned with mild, non-abrasive solutions (such as diluted bleach, ammonia, or alcohol solutions).

The surface finish may be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.

Do not use steel wool or other abrasive materials.

6 Environmental Specifications

Temperature range	Operating	0°C~+40°C
	Storage	-40°C~+70°C
Humidity range	Operating	15% - 95 % ,non-condensing
	Storage	10% - 95 %, non-condensing
Altitude range	Operating	-500 ~+4,600 m (57.0kPa~106.0kPa)
	Storage	-500 ~ + 11,175 m (50.0kPa~106.0kPa)